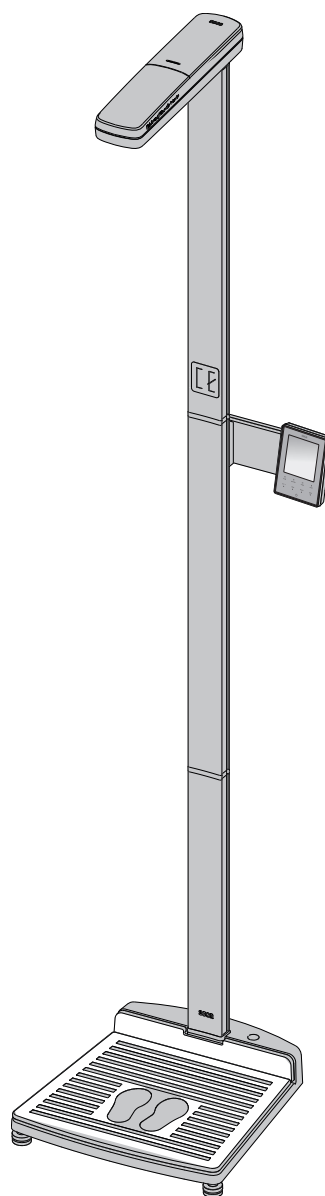
seca 287/286

Instructions for use

17-10-07-671-100b_2024-11S

Deutsch – Gebrauchsanweisung	3
English – Instructions for use	47
Français – Mode d’emploi.....	90
Italiano – Istruzioni per l’uso	135
Español – Instrucciones de uso	179
Português – Manual de instruções de utilização	224
Polski – Instrukcja obsługi.....	269

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TABLE OF CONTENTS

1 About this document	48	Displaying measuring result permanently (hold).....	72
1.1 Representation in text	48	Entering patient data (input)	72
1.2 Representation in diagrams	48	Entering patient's gender	72
2 Description of device	48	Determining Body Mass Index (BMI).....	73
2.1 Intended use	48	Determining Body Fat Ratio (BFR)	73
2.2 Contraindications	48	Clearing stored values (clear)	73
2.3 Clinical benefit	49	Switching weighing range automatically ..	74
2.4 Patient target group.....	49	Switch off the device	74
2.5 User qualification.....	49	6.2 Configuring voice output.....	74
Administration/network operation	49	Selecting language (LAn9).....	74
Operation.....	49	Setting volume (VOL).....	74
Assembly	49	Activating/deactivating announcement of measuring results (reSUL)	74
2.6 Functional description	49	Activating/deactivating the announcement "Take your printed ticket" (Print)	75
Measuring weight and height	49	Activating/deactivating announcement of patient instructions (InStr)	75
External interface module (optional accessory)	49	Configuring beeps	75
seca connect 103 software (optional accessory)	49	6.3 Other functions (menu).....	76
3 Safety information	49	Deleting values automatically (ACLR).....	76
3.1 Safety information in these instructions for use.....	49	Deactivating height measurement.....	76
3.2 Basic safety information	50	Switching between BMI and BFR (bodY) ..	76
Handling device.....	50	Setting display backlighting (LCd).....	77
Preventing electric shock	51	Using correction value for height measurement (ZErO)	77
Preventing injuries and infections.....	51	Saving additional weight permanently (Pt) ..	78
Preventing damage to device.....	52	Activating Autohold (AHOLd) function	78
Handling measuring results.....	53	Activating/deactivating beeps (bEEP)	78
Handling packaging material.....	53	Setting filtering (FIL).....	79
4 Overview	54	Non-verified models: Switching weight unit (Unit)	79
4.1 Controls	54	Switching unit of height (HUnit).....	80
4.2 Display controls	55	Restoring factory settings (rESet)	80
4.3 Labels	56	7 Hygiene treatment	81
4.4 Symbols in the display	58	7.1 Cleaning	81
4.5 Menu structure	59	7.2 Disinfecting.....	81
4.6 Navigating in the menu.....	60	7.3 Sterilizing.....	82
5 Starting up device	60	8 Function check	82
5.1 Scope of delivery	60	9 Troubleshooting	82
5.2 Assembling device	61	10 Servicing	85
Introduction	61	10.1 Weight measurement, verified models.....	85
Fitting second column element	61	10.2 Weight measurement, non-verified models ..	86
Fitting multifunctional display.....	62	10.3 Height measuring devices.....	86
Fitting third column element.....	63	11 Technical data	86
Fitting the ultrasound head.....	64	11.1 General technical data	86
Attaching the quick guide.....	65	11.2 Weight measurement	87
5.3 Setting up device.....	66	11.3 Height measurement	88
5.4 Transporting device.....	67	12 Optional accessories and spare parts	88
5.5 Connecting power supply unit	68	13 Disposing of the device	88
5.6 Calibrating height measurement	69	14 Warranty	89
5.7 Disinfecting device	69	15 Declaration of conformity	89
6 Operation	70		
6.1 Measure.....	70		
Switch on device	70		
Perform the measurement.....	70		
Activating/deactivating key lock.....	71		
Weighing babies/toddlers (2 in 1)	71		

1 ABOUT THIS DOCUMENT

1.1 Representation in text

Symbol	Description
✓	Requirement for actions
►	Action
1. 2.	Actions with specified sequence
a) b)	Steps of an action with specified sequence
⇒	Result of an action
• •	First level of a list
– –	Second level of a list

1.2 Representation in diagrams

Symbol	Description
	Indicates directions of movement of the device or of device components
	Navigation path in menu displays
	• Correct action • Correct result of action
	• Incorrect action • Incorrect result of action
	End of a procedure, e.g. the installation of a part

2 DESCRIPTION OF DEVICE

2.1 Intended use

The electronic measuring station supports physicians in decision-making regarding diagnosis or therapy, based on the patient's weight, height or changes of the measured parameters.

2.2 Contraindications

No contraindications are known.

2.3 Clinical benefit

The electronic measuring station supports physicians in decision-making regarding diagnosis or therapy, based on the patient's weight and height.

By that, the electronic measuring station only provides an indirect clinical benefit.

The physician must perform additional examinations for an accurate diagnosis.

2.4 Patient target group

The device is intended for persons of any age who can stand independently and who do not exceed the maximum weight capacity of the device. Babies and toddlers who cannot stand independently can be weighed using the **2 in 1** function whilst being held by an adult who can stand independently.

2.5 User qualification

Administration/network operation	The device may only be set up and incorporated in a network by experienced administrators or hospital technicians.
Operation	The device may only be operated by persons with formal training in the healthcare sector or in medicine.
Assembly	Devices shipped partly assembled may only be assembled by sufficiently qualified persons (such as specialist dealers, hospital technicians or seca Service technicians).

2.6 Functional description

Measuring weight and height	Weight is recorded by four load cells. The height is measured using ultrasound. Body Mass Index (BMI) or Body Fat Ratio (BFR) can be calculated automatically from the measuring results. The device guides the patient through the measuring process with configurable voice output.
External interface module (optional accessory)	Data can be sent to the seca connect 103 software via a WiFi or LAN connection with the aid of the seca 452 external interface module (optional).
seca connect 103 software (optional accessory)	The seca connect 103 software obtains measurement data from an internal or external interface module and sends them to compatible seca measuring devices or to third-party EMR systems.

3 SAFETY INFORMATION

3.1 Safety information in these instructions for use

DANGER!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

WARNING!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

**CAUTION!**

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

NOTICE!

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measuring results may be incorrect.

NOTE

Includes additional information about use of the device.

3.2 Basic safety information

Handling device

- ▶ Please take note of the information in these instructions for use.
- ▶ Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.
- ▶ In the interest of patient safety, you and your patients are obliged to report serious events that occur in connection with this product to the manufacturer and to the authority responsible in your country.

**DANGER!****Risk of explosion**

- ▶ Do not use the device in an environment in which one of the following gases has accumulated:
 - oxygen
 - flammable anesthetics
 - other flammable substances/air mixtures

**CAUTION!****Patient hazard, damage to device**

- ▶ Additional devices connected to medical electrical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the requirements of standards for medical systems (see IEC 60601-1-1 or Section 16 of edition 3.1 of IEC 60601-1 respectively). Anyone connecting additional devices to medical electrical devices is considered a system configurator and therefore responsible for ensuring that the system complies with the requirements of standards for systems. This also applies to additional devices recommended by seca. Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.

**CAUTION!****Patient hazard, damage to device**

- ▶ Have servicing carried out regularly as described in the relevant section of this document.
- ▶ Technical modifications may not be made to the device. The device does not contain any user-serviceable parts. Only have servicing and repairs performed by an authorized seca service partner. You can find a service partner in your vicinity at www.seca.com or by emailing service@seca.com.
- ▶ Use only seca original accessories and spare parts, otherwise seca will not grant any warranty.

**CAUTION!****Patient hazard, malfunction**

- ▶ Keep other medical electrical devices, e.g. high-frequency surgical devices, a minimum distance of approx. 1 meter away to prevent faulty measurements or wireless transmission interference.
- ▶ Keep HF devices such as cellphones at a minimum distance of approx. 1 meter to prevent faulty measurements or wireless transmission interference.
- ▶ The actual transmission output of HF devices may require minimum distances of more than 1 meter. For details, go to www.seca.com.

Preventing electric shock**WARNING!****Electric shock**

- ▶ Set up devices which can be operated with a power supply unit so that the power supply socket is within easy reach and the power supply can be disconnected quickly.
- ▶ Ensure that your local electricity supply matches the details on the power supply unit.
- ▶ Never touch the power supply unit with wet hands.
- ▶ Do not use extension cables and multiple outlets.
- ▶ Ensure that cables are not crushed or damaged by sharp edges.
- ▶ Ensure that cables do not come into contact with hot objects.
- ▶ Do not operate the device at an altitude of more than 3000 m above sea level.

Preventing injuries and infections**WARNING!****Injury from falling**

- ▶ Ensure that the device is steady and level.
- ▶ Route connector cables (if present) so that neither users nor the patient can trip over them.
- ▶ Assist people with limited mobility, e.g. when they are getting up from a wheelchair.
- ▶ Ensure that the patient does not step straight onto or off the edges of the patient standing area.
- ▶ Ensure that the patient steps onto and off the patient standing area slowly and safely.

**WARNING!****Risk of slipping**

- ▶ Ensure that the patient standing area is dry before the patient steps onto it.
- ▶ Ensure that the patient has dry feet before stepping onto the patient standing area.
- ▶ Ensure that the patient steps onto and off the patient standing area slowly and safely.

**CAUTION!****Injury, damage to device**

The patient standing area consists of a glass plate. Damage (e.g. scratches, cracks or chips) represent a risk of injury. Damage can also lead to the glass plate breaking.

- ▶ Do not put any sharp-edged objects on the glass plate.
- ▶ Before using each time, check the glass plate for scratches, cracks and chips. If you find damage of this kind, have the glass plate replaced with a new one.
- ▶ Do not use the device if the glass plate is damaged.

Preventing damage to device

WARNING!

Risk of infection

- ▶ Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- ▶ Subject the device to a hygiene treatment at regular intervals as described in the relevant section of these instructions for use.
- ▶ Ensure that the patient does not have any infectious diseases.
- ▶ Ensure that the patient does not have any open wounds or infectious skin alterations which may come into contact with the device.

NOTICE!

Damage to device

- ▶ Ensure that fluids never get inside the device. These can destroy the electronics.
- ▶ For devices with power supply operation: Switch off the device before you disconnect the power supply unit from the power supply socket.
- ▶ For devices with power supply operation: If the device is not to be used for an extended period, disconnect the power supply unit from the power supply socket. Only then is the device de-energized.
- ▶ For devices with battery or rechargeable battery operation: If you are not using the device for an extended period of time, remove batteries or rechargeable batteries. Only then is the device de-energized.
- ▶ Do not drop the device.
- ▶ Do not subject the device to shocks or vibrations.
- ▶ Perform a function check before each use as described in the corresponding section in this document. Do not operate the device if it is not working properly or is damaged.
- ▶ Do not place the device in direct sunlight and ensure that it is not placed in the direct proximity of a heat source. The excessive temperatures could damage the electronics.
- ▶ Avoid rapid temperature fluctuations. If the device is transported so that a temperature difference of over 20 °C occurs, the device must be left to stand for at least 2 hours before it is switched on, otherwise condensation may form; this may damage the electronics.
- ▶ Use the device only in the intended ambient conditions.
- ▶ Store the device only in the intended storage conditions.
- ▶ Use only cleaning agents and disinfectants which match the details in the section entitled "Hygiene treatment".
- ▶ For scales: Ensure that maximum capacity is not exceeded.

Handling measuring results

WARNING!

Patient hazard

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (weight: kilograms/grams, height: meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- ▶ Only use measuring results in SI units.
- ▶ The user takes sole responsibility for the use of measuring results in non-SI units.

NOTICE!

Inconsistent measuring results

- ▶ Before you save and continue using measured values determined using this device (e.g. in seca software or in an EMR system), ensure that the measured values are plausible.
- ▶ If measured values have been sent to seca software or to an EMR system, ensure before continuing to use them that the measured values are plausible and assigned to the correct patient.

NOTICE!

Malfunction caused by other ultrasonic emitters

If there are other ultrasonic emitters in the immediate vicinity of the device - automatic door openers, for example - incorrect measurements will result.

- ▶ Ensure that there are no other ultrasonic emitters in the same room or in the immediate vicinity of the device.

NOTICE!

Faulty measurement due to reflections

If there are objects or people in the immediate vicinity of the device, incorrect measurements will result.

- ▶ Ensure that there are no objects or people within 0.5 meters of the front or side of the scale during the measurement procedure.
- ▶ Ensure that the device is at least 0.2 meters away from the wall.
- ▶ Ensure that the patient is not wearing any kind of hair accessory on top of their head.

Handling packaging material

WARNING!

Risk of suffocation

Packaging materials made of plastic film (bags) present a risk of suffocation.

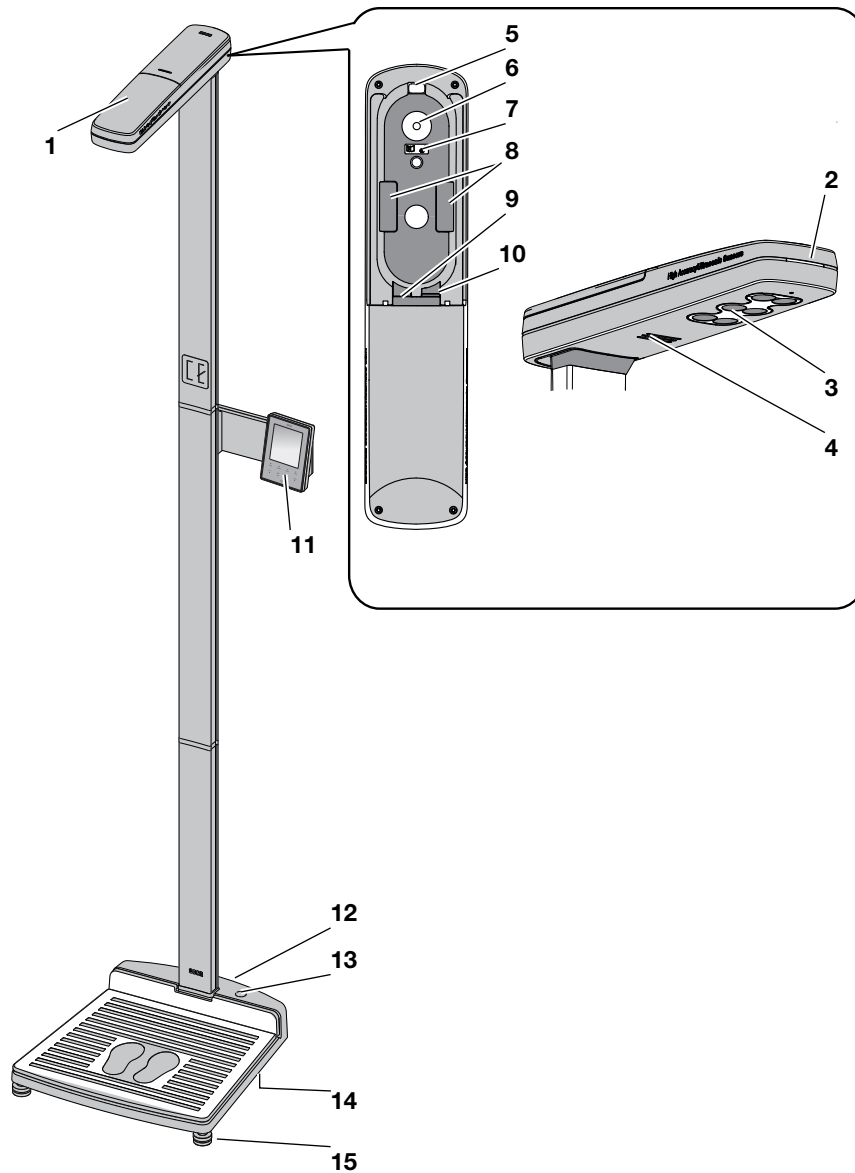
- ▶ Store packaging material out of the reach of children.
- ▶ If the original packaging material is no longer available, only use plastic bags with safety holes to reduce the risk of suffocation. Use recyclable materials if possible.

NOTE

Store the original packaging material for future use (e.g. returning for servicing).

4 OVERVIEW


4.1 Controls

Item	Control	Function
1	Ultrasound head	For measuring height
2	Status LED	Indicates the status of the measurement procedure
3	Ultrasonic sensors	For measuring height
4	Loudspeaker	For voice output
5	Cable duct	For threading the connector cable into the ultrasound head
6	Assembly opening	For assembling the measuring head on the column
7	Notice	Follow assembly notes in the instructions for use
8	Cable brackets	For securing the connector cable

Item	Control	Function
9	Service interface	For use by seca Service
10	Connector socket	For connecting the ultrasound head to the multifunctional display
11	Multifunctional display	Central control and display element
12	Casters	For transporting short distances
13	Spirit level	Indicates whether the device is horizontal
14	Power supply connection	For connecting the power supply unit
15	Foot screws	For precise alignment

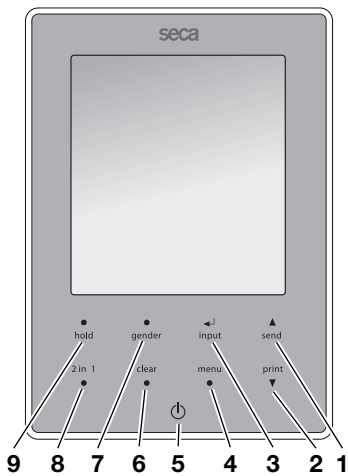4.2 Display controls

Item	Control	Function
1	 send	- Send measuring results to an EMR system via seca connect 103 software (condition: seca 452 interface module, available as an option) - In the menu: - Select submenu, select menu item - Increase value (press briefly: Value is increased by 1 increment, press and hold: Value is increased until key is released)
2	print 	- In the menu: - Select submenu, select menu item - Reduce value (press briefly: Value is reduced by 1 increment, press and hold: Value is reduced until key is released)
3	 input	- During weighing: - Enter patient data (age, gender, PAL) - In the menu: - Confirm selected menu item - Save set value
4	menu 	- During weighing: - Call up multifunctional display menu - In the menu: - Press briefly: Go back one menu level - Press and hold: Exit menu
5		Switch the device on and off

Item	Control	Function
6	clear ●	For deleting data (patient data, height, BMI , BFR) entered manually or received wirelessly
7	● gender	Enter the patient's gender
8	2 in 1 ●	Start the 2 in 1 function for weighing babies and toddlers
9	● hold	Activate the Hold function

4.3 Labels

Markings on the device and on the type plate	
Symbol	Meaning
	Name and address of manufacturer, date of manufacture
UDI	Unique Device Identifier (product identification number)
	Article number
	Serial number
GAL	Value in m/s ² (model-dependent) • Gravitational acceleration on earth • Depends on the intended location
ProdID	Product identification number
Approval Type	Type designation of design approval
	Follow instructions for use
	Medical electrical device, Type B
IP20	Type of protection in accordance with IEC 60529: • Protection against ingress of solid foreign bodies with a diameter of over 12.5 mm • Protection against access with fingers • No protection from ingress of water
e	Value in units of mass (verified models) Used to classify and verify a scale
d	Value in units of mass (non-verified models) States the difference between two consecutive display values
→ x ←	Active weighing range
	Class III scale to 2014/31/EU


17-10-07-671-100b_2024-11S

Markings on the device and on the type plate	
Symbol	Meaning
	Device complies with EU directives • M: Conformity label in compliance with directive 2014/31/EU for non-automatic scales (verified models) • 24: (Example: 2024) Year in which declaration of conformity was implemented and the CE label was applied (verified models) • 0102: Notified Body for Metrology (verified models) • 0123: Notified Body for Medical Devices
	Medical device in accordance with Regulation (EU) 2017/745
	Device meets the requirements of the USA and Canada. Certified and tested by a licensing laboratory (NRTL) of TÜV SÜD Product Services GmbH.
	Device complies with United Kingdom directives • M: Conformity label in compliance with UK directive SI 2016 no. 1152 for non-automatic scales (NAWIR)(verified models) • 24: (Example: 2024) Year in which declaration of conformity was implemented and the UKCA label was applied (verified models) • xxxx: Notified Body for Medical Devices of the United Kingdom • yyyy: Notified Body for Metrology of the United Kingdom (verified models)
	Importer/representative in the United Kingdom: seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom
	Importer/representative in Switzerland: seca ag (schweiz) Medizinische Waagen und Messsysteme Schönematt Str. 2 CH-4153 REINACH
	Type plate on the power supply connection socket • Power supply voltage required in V • Maximum current consumption in mA • : Note polarity of device connector • : Operate device with direct current • : Use compatible seca power supply units only
	Do not dispose of device in household waste

Markings on the packaging	
Symbol	Meaning
	Protect from moisture
	Arrows indicate top of product Transport and store in an upright position
	Fragile, do not throw or drop


Markings on the packaging	
Symbol	Meaning
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. humidity for transport and storage
	Permitted min. and max. air pressure for transport and storage
	Open packaging here
	Packaging material can be disposed of through recycling programs

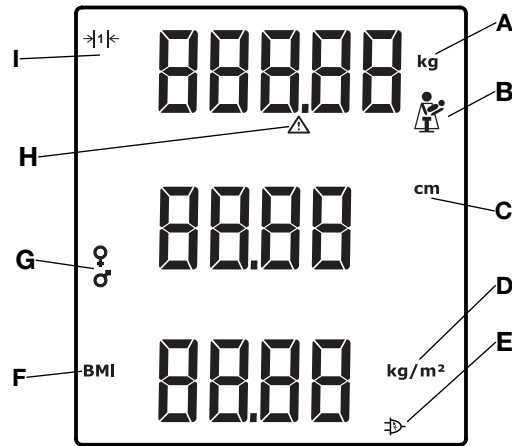4.4 Symbols in the display

Item	Symbol	Meaning
A	kg	Unit in which weight is displayed
B		2 in 1 function active for weighing babies and toddlers
C	cm	Unit in which height is displayed
D	kg/m ²	Unit in which BMI is stated
E		Operation with power supply unit
F	BMI	Indicates that Body Mass Index (BMI) will be calculated automatically
G		Patient's gender
H		Non-verified function active (for technical reasons, symbol is displayed even on non-verified scales)
I	→ ←	Active weighing range

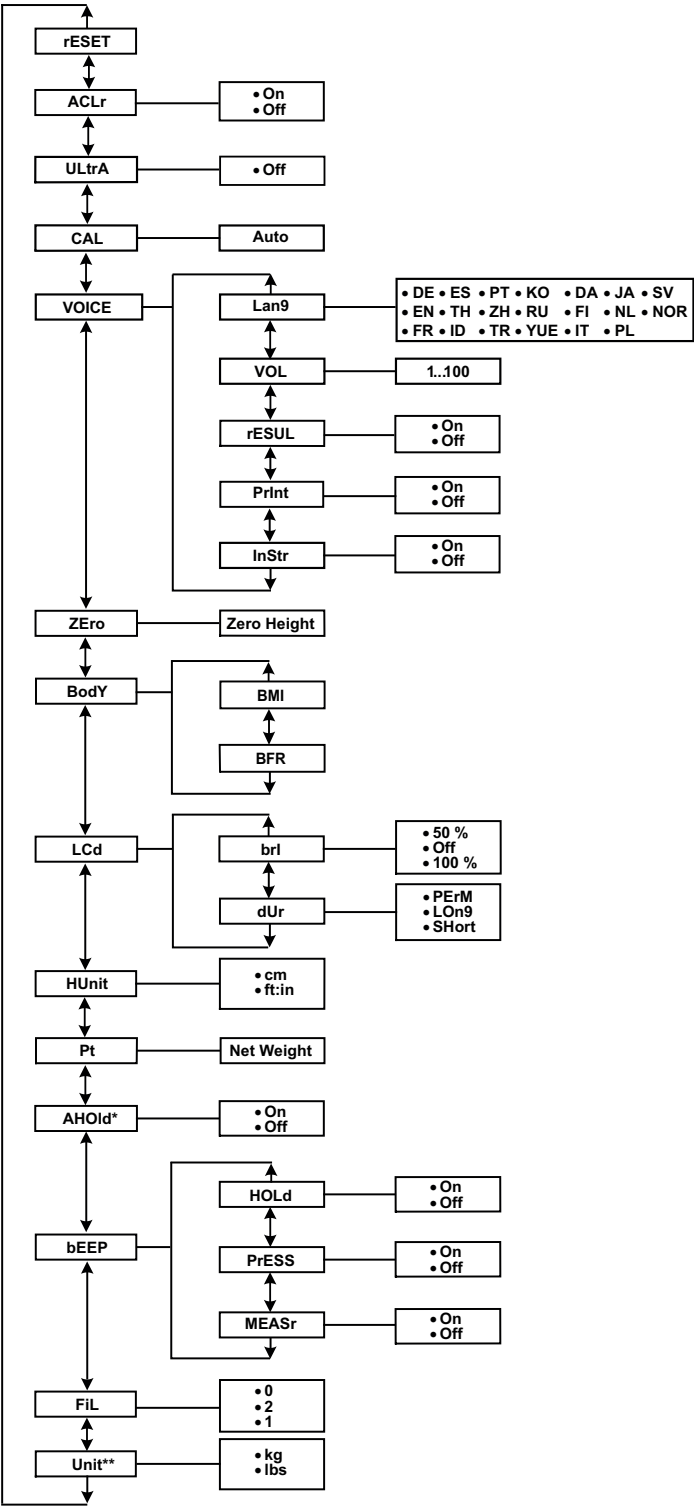
4.5 Menu structure

AClr	Autoclear
ULtrA	Ultrasound Measurement
CAL	Calibration
Lan9	Language
VOL	Volume
rESUL	Results
InStr	Instructions
BMI	Body Mass Index
BFR	Body Fat Ratio
LCd	Liquid Crystal Display
brl	Brightness
dUr	Duration
PErM	Permanent
LOn9	Long
HUnit	Height Unit
Pt	Pre-Tare
AHOld	Autohold
MEASr	Measure
FIL	Filter

* For measuring weight only
** Non-verified models

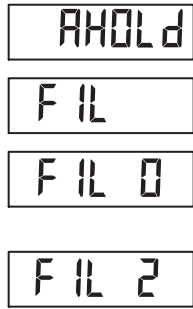4.6 Navigating in the menu

- 
1. Press the  key of the multifunctional display.
 2. Press the **menu** key.
⇒ The last menu item selected is displayed (in this case: Autohold **AHOLD**).
 3. Press the **▲ (send)** or **▼ (print)** key until the desired menu item is displayed (in this case: filtering **FIL**).
 4. Confirm the selection by pressing the **input** key.
⇒ The current setting for the menu item or a submenu are displayed (in this case: level **0**).
 5. To change the setting, press the **▲ (send)** or **▼ (print)** key until the desired setting is displayed (in this case: level **2**).
 6. Confirm the setting by pressing the **input** key.
⇒ The setting is adopted.
⇒ The menu closes.

NOTE

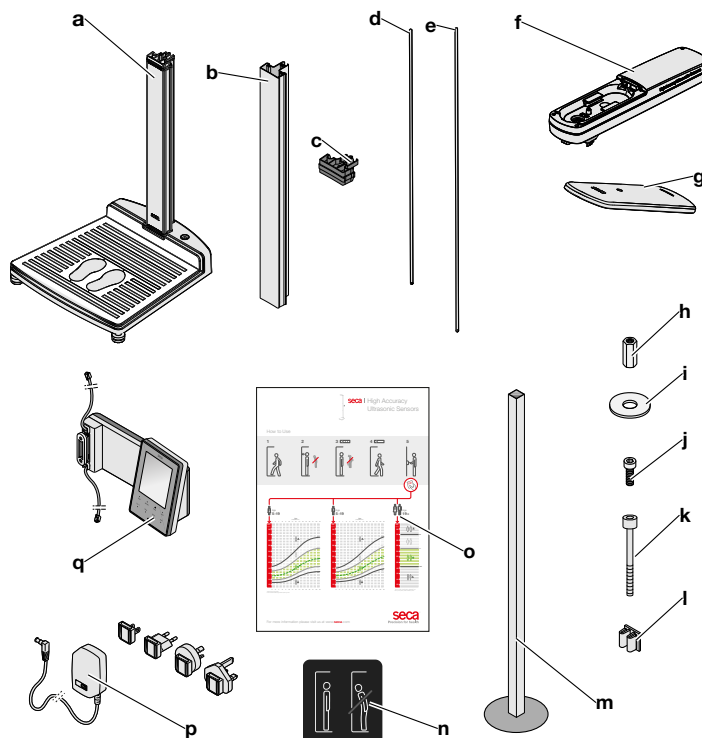
Briefly press the **menu** key to go back one menu level.

Press and hold the **menu** key to exit the menu at any time.

If no key is pressed for about 30 seconds, you exit the menu automatically.

5 STARTING UP DEVICE

5.1 Scope of delivery


Item	Component	Pcs.
a	Lower part of device, preassembled • Weighing platform • First column element	1
b	Column element	2
c	Column connector	1
d	Threaded rod, short	1
e	Threaded rod, long	1
f	Ultrasound head	1
g	Ultrasound head cover	1
h	Coupling nut	1
i	Plain washer	1
j	Hex head socket screw, multifunctional display	2
k	Hex head socket screw, ultrasound head cover	1
l	Cable clip	5
m	Calibration rod	1
n	Label: "Correct posture"	1
o	Poster: "Quick guide to measuring"	1
p	Power supply unit	1
q	Multifunctional display with connector cables	1
–	Instructions for use	1
–	Tool kit: • Hex socket wrench, 2 mm • Hex socket wrench, 5 mm • Open-ended spanner, 10 mm	1


5.2 Assembling device

Introduction The first column element is fitted as standard to the lower part of device on delivery.

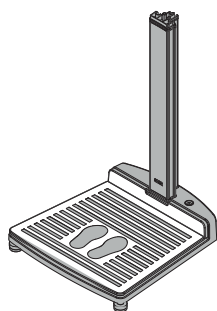- Use an assistant to perform the assembly.

Fitting second column element

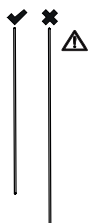NOTICE!

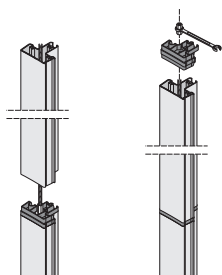
Damage to device as a result of assembly errors

If the long and short threaded rods are mixed up, it will not be possible to assemble the device so that it is fit for purpose.

- Ensure that the short threaded rod is fitted when assembling the second column element.

1. Screw the short threaded rod (approx. 3 Nm) into the coupling nut of the first column element.

2. Push a column element over the threaded rod onto the first column element.
3. Place the column connector on the second column element so that the threaded rod protrudes out of the bore in the column connector.
4. Screw a coupling nut onto the threaded rod.
5. Tighten the coupling nut.

Fitting multifunctional display

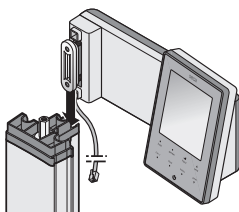
The multifunctional display is suspended in the groove of the column elements and fixed in position with a screw clamp.

NOTICE!


Malfunction as a result of reflecting multifunctional display

If the multifunctional display is fitted to the first column element, faulty measurements will result.


► Fit the multifunctional display to the second column element.

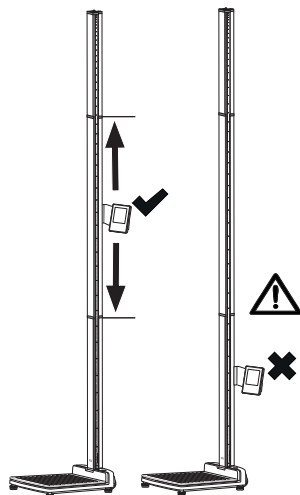1. Insert the sliding block of the multifunctional display in the groove of the second column element.

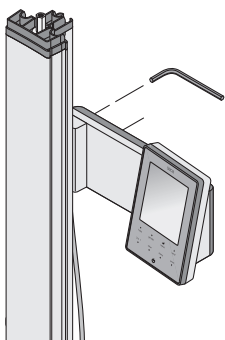2. Slide the multifunctional display until it is at the right height for you.


3. Tighten the two hex head socket screws.
4. Route the connector cable of the multifunctional display in the groove of the column elements as far as the lower part of device.
5. Draw the connector cable through the bore of the lower part of device.

NOTICE!


Malfunction as a result of assembly errors

If the cables are fitted under severe mechanical load, faulty displays and failure of the display may result.

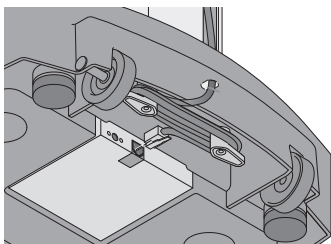
- Route all cables to prevent sharp bends and kinked connectors.

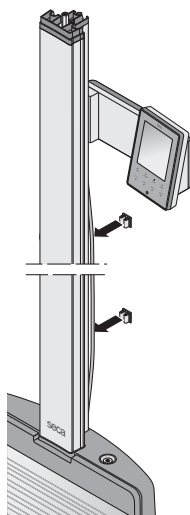6. Wind the free end of the connector cable onto the cable store in the lower part of device.
7. Plug the connector of the connector cable into the appropriate socket of the weighing platform.

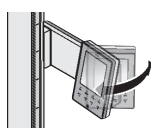
8. Fix the connector cable in the groove of the column elements using the cable clips.

9. Swivel the multifunctional display so that it is convenient for you to read.

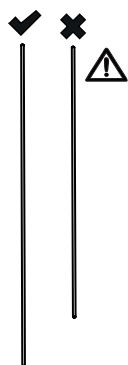Fitting third column element


NOTICE!

Damage to device as a result of assembly errors

If the long and short threaded rods are mixed up, it will not be possible to assemble the device so that it is fit for purpose.

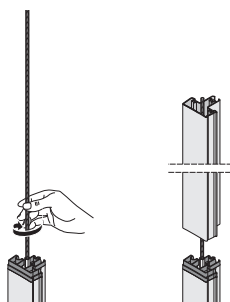- Ensure that the long threaded rod is fitted when assembling the third column element.

1. Screw the long threaded rod hand-tight (approx. 3 Nm) into the coupling nut of the second column element.
2. Push the column element over the threaded rod onto the column connector of the second column element.

Fitting the ultrasound head

The ultrasound head is fitted to the third column element and connected to the connector cable of the multifunctional display.

NOTICE!

Damage to device as a result of assembly errors

The third column element sits loose on the second column element during assembly.

- Hold the third column element firmly as you lay the device on the floor.
- Ensure that the third column element is in the right position before you fit the ultrasound head.

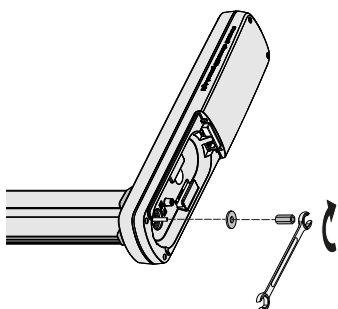
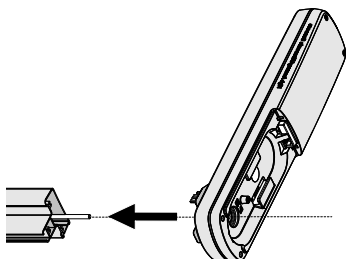
NOTICE!

Malfunction as a result of faulty multifunctional display

When the device is laid down, the multifunctional display is directly on the floor and may be damaged.

- Lay the device down slowly and carefully on a soft surface, a blanket, for example.

1. Tilt the device and carefully lay it on the floor.
2. Push the ultrasound head onto the free end of the threaded rod.


3. Put the plain washer onto the threaded rod.
4. Screw the coupling nut onto the threaded rod (3 Nm).
5. Tighten the coupling nut (3 Nm).

NOTICE!


Damage to device and malfunction as a result of assembly errors

If the coupling nut in the ultrasound head is not tightened enough, the column elements will be too loosely connected. The device will be unstable and faulty measurements will occur.

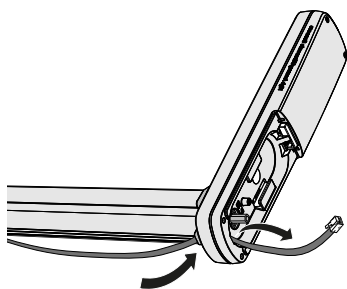- Tighten the coupling nut to the specified torque so that the columns are located firmly on one another and there is no gap between the column connector and the third column element.

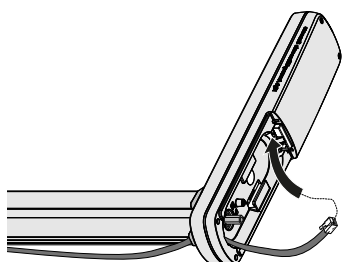6. Push the connector cable of the multifunctional display through the cable duct in the ultrasound head.


7. Plug the connector cable into the left-hand modular socket.


NOTICE!

Damage to device and malfunction as a result of trapped connector cable

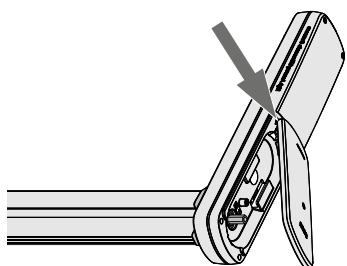- Before attaching the cover, route the connector cable in the cable compartment in such a way that it cannot be trapped between the cover and the cable compartment.
- If the connector cable is too long, loop it into the cable compartment.

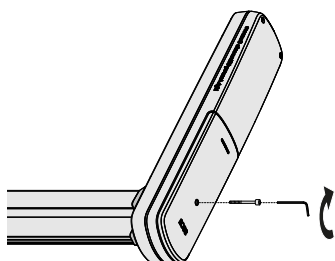8. Put on the ultrasound head cover.


9. Tighten the ultrasound head cover (2 Nm).
10. Carefully return the device to an upright position.

Attaching the quick guide

The quick guide poster shows the patient the sequence of the measurement procedure.

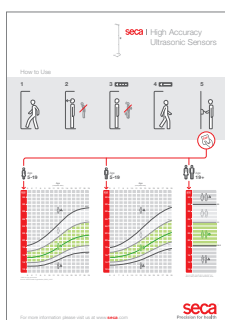- Place the "Quick guide to measuring" poster in a clearly visible location close to the device.

5.3 Setting up device

CAUTION!

Faulty measurement as a result of devices being set up incorrectly

Soft floors (wooden boards, for example) give under the patient's weight and falsify the measuring result. Incorrect ambient conditions or lack of alignment lead to measurement errors.

- ▶ Select a setup location with a level, stable floor in order to achieve precise measuring results.
- ▶ Use the device only in the intended ambient conditions outlined in the section entitled "Technical data".
- ▶ Align the device before using it and every time it changes location.

CAUTION!

Injury from a lack of stability

If the foot screws are screwed out too far, they may come loose from the device. The device will then be unstable.

- ▶ Screw the foot screws out a maximum of 10 mm.
- ▶ If the device cannot be aligned horizontally with the screws screwed out as far as possible, the setup location is unsuitable. Set the device up in a suitable location.

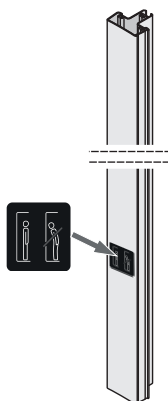
NOTICE!

Faulty measurement

Under certain conditions, part of the patient weight will not be included in the measurement. The measuring result is then incorrect.

- ▶ Set up the device so that only its foot screws are in contact with the ground.
- ▶ Set up the device so that the weighing surface is not in contact with any other objects.
- ▶ Set up the device so that the glass plate is in free contact with the lower part of device and is not interfered with by other objects, e.g. a cable.
- ▶ During weighing, ensure that the patient does not touch the column of the device.

1. Place the device on a firm, level surface.
2. Affix the "Correct posture" label to the third column element.


NOTICE!

Malfunction caused by other ultrasonic emitters

If there are other ultrasonic emitters in the immediate vicinity of the device - automatic door openers, for example - incorrect measurements will result.

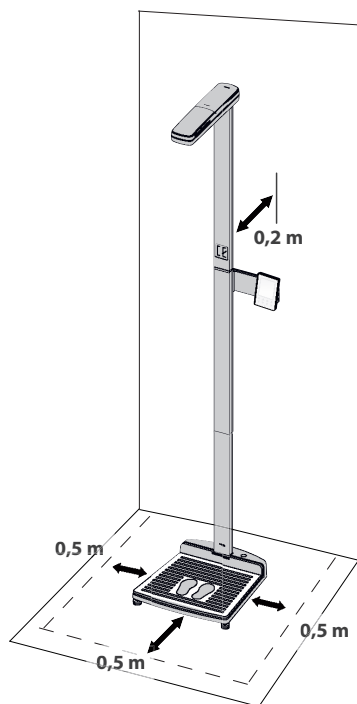
- ▶ Ensure that there are no other ultrasonic emitters in the same room or in the immediate vicinity of the device.

**NOTICE!****Faulty measurement due to reflections**


If there are objects or people in the immediate vicinity of the device, incorrect measurements will result.

- ▶ Ensure that there are no objects or people within 0.5 meters of the front or side of the scale during the measurement procedure.
- ▶ Ensure that the device is at least 0.2 meters away from the wall.
- ▶ Ensure that the patient is not wearing any kind of hair accessory on top of their head.

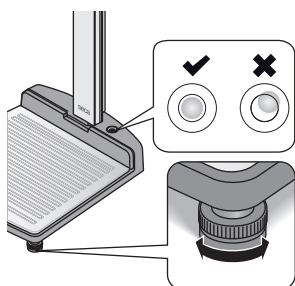3. Mark the area shown in the illustration e.g. using colored adhesive tape.

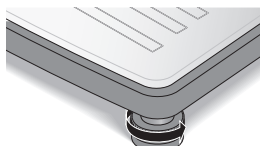4. Undo the knurled wheels.
5. Align the scale by turning its foot screws so that the air bubble of the spirit level is in the exact center of the circle.

6. Tighten the knurled wheels in the direction of the arrow.
⇒ The foot screws are secured against being adjusted.
7. Tighten the knurled wheels.
⇒ The foot screws are secured against being adjusted.

5.4 Transporting device


The device has two casters that facilitate transport over short distances.

**CAUTION!****Risk of injury and damage to device**

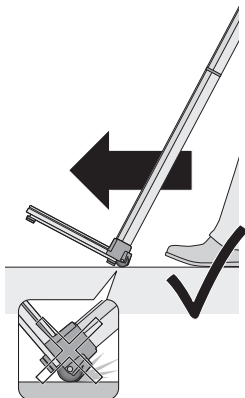The device must be tilted for transport. If the device is tilted without care, this may lead to injuries and damage to the device.

- ▶ Ensure that there are no other persons in the immediate vicinity.
- ▶ Ensure that there are no objects in the immediate vicinity.

1. Disconnect the power supply unit from the socket.

2. Tilt the device until it can be moved freely on the casters.
3. Transport the device to its new setup location.
4. Return the device to an upright position.
5. Plug the power supply unit into a power supply socket.
6. Align the device → [Setting up device, page 66](#).
7. Perform a function check using → [Function check, page 82](#).

5.5 Connecting power supply unit

The connection for the power supply unit is located on the underside of the lower part of device. To establish the power supply for the scale and multifunctional display, proceed as follows:

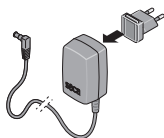
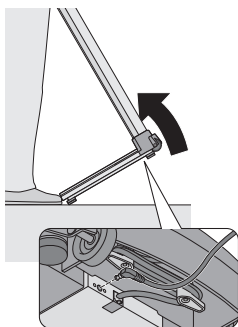
WARNING!

Personal injury or damage to the device as a result of incorrect power supply units

Conventional power supply units may deliver a higher voltage than is indicated on them. The measuring device may overheat, catch fire, melt or short-circuit.

- Use only original seca power supply units with a controlled 12 V output voltage.

1. Tilt the device forward.
2. Insert the device connector of the power supply unit into the connecting socket of the scale.
3. Carefully return the device to an upright position.

4. Insert the connector required for your power supply into the power supply unit.
5. Plug the power supply unit into a power supply socket.

5.6 Calibrating height measurement

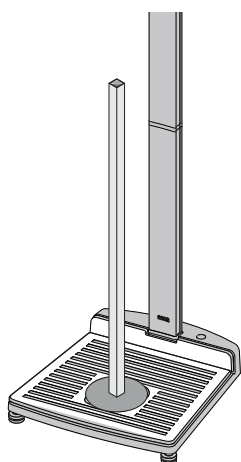
You can perform automated calibration with the aid of the calibration rod supplied.

NOTE

Perform calibration at least once a year to achieve consistently reliable measuring results.

CAL

Auto

rEAdY

1. Ensure that there is no load on the scale.
2. Switch on the device.
3. From the menu, select the **CAL** menu item.
4. Confirm your selection.
5. Select the **Auto** menu item.
6. Confirm your selection.
⇒ The **CAL 0.0** message is displayed.
7. Wait until the message **CAL 81,5** is displayed.
8. Place the supplied calibration rod onto the foot pictogram of the weighing platform.

NOTE

Ensure that there are no objects or people in the immediate vicinity of the device during the calibration process → [Setting up device, page 66](#)

9. Wait until the message **rEAdY** is displayed.
10. Remove the calibration rod from the lower part of device.
⇒ The device restarts automatically and is then ready to measure.

5.7 Disinfecting device

- Disinfect the following components **before** initial commissioning (→ [Disinfecting, page 81](#)):
 - Weighing platform
 - Column
 - Multifunctional display
 - Ultrasound head

6 OPERATION

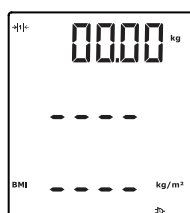
CAUTION!

Personal injury/faulty measurement

- Before using the device each time, perform a function check as described in the corresponding section of these instructions for use.

6.1 Measure

Switch on device

1. Ensure that there is no load on the scale.
2. Press the  key of the multifunctional display.
 - ⇒ All the elements of the display are shown briefly.
 - ⇒ **SECA** appears in the display.
 - ⇒ The scale is operational when **0.00** appears in the display.

Perform the measurement

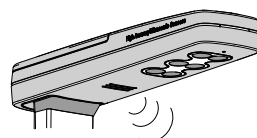
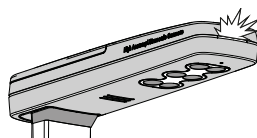
The measuring sequence described below is based on the factory settings. Configuration options can be found here: → [Configuring voice output, page 74](#), → [Other functions \(menu\), page 76](#).

NOTICE!


Faulty measurement due to reflections

If there are objects or people in the immediate vicinity of the device, incorrect measurements will result.

- Ensure that there are no objects or people within 0.5 meters of the front or side of the scale during the measurement procedure.
- Ensure that the device is at least 0.2 meters away from the wall.
- Ensure that the patient is not wearing any kind of hair accessory on top of their head.

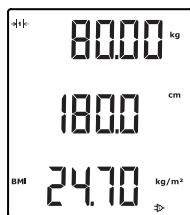1. Ensure that the status LED on the ultrasound head is on.
2. If available, attach the privacy cover to the multifunctional display (→ [Optional accessories and spare parts, page 88](#)) to prevent unauthorized people from reading the measuring results.
3. Ask the patient to step onto the weighing platform.
4. Make sure that the patient steps forward onto the weighing platform and adopts an upright posture.
5. Ensure that the patient is standing on the marked area (foot pictogram) of the weighing platform.
6. Ask the patient to follow the instructions given by the device.

7. Read off the measuring result and take the printed ticket.

NOTE

You will need a seca wireless printer to print out the measuring result. A description of how to connect a seca wireless printer can be found here: [Setting up seca 360° wireless connection](#).

Activating/deactivating key lock

Use the key lock to prevent undesired settings being made to the multifunctional display once the device has been switched on.

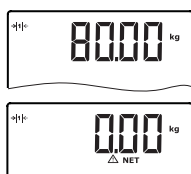
NOTE

No separate symbol for an activated key lock is displayed in the multifunctional display.

Key lock cannot be activated while you are navigating in the menu.

1. Ensure that the device is switched off.
2. Press the  key of the multifunctional display.
3. Wait until **SECA** is displayed.
4. Press the **menu** key of the multifunctional display.
⇒ The key lock is activated.
5. To deactivate the key lock, press the  key of the multifunctional display again.
6. Wait until **SECA** is displayed.
7. Press the **menu** key of the multifunctional display.
⇒ The key lock is deactivated.

Weighing babies/toddlers (2 in 1)

The **2 in 1** function can be used to determine the weight of babies and toddlers. The child is held in an adult's arms during weighing to do this. Proceed as follows:

1. Ensure that there is no load on the scale.
2. Ask the adult to stand on the scale.
⇒ The adult's weight is displayed.
3. Press the **2 in 1** key.
⇒ The adult's weight is stored as the initial weight.
⇒ **0.00**, the  symbol and the **NET** message are displayed.

NOTICE!

Faulty measurement as a result of initial weight changing

If the child is weighed together with a different adult, it will not be possible to determine the child's weight correctly.

- Ensure that the child is always weighed with the adult with whom the initial weight was determined.
- Ensure that the adult's weight does not change - as a result of their taking off a piece of clothing, for example.

4. Ask the adult to get onto the scale holding the child.
⇒ The child's weight is displayed.
⇒ The symbols  and  as well as the messages **HOLD** and **NET** are displayed.
5. To deactivate the function, press the **2 in 1** key or switch off the scale.

Displaying measuring result permanently (hold)

If you activate the **Hold** function, the measuring result continues to be displayed once the weight has been removed from the scale. This enables you to attend to the patient before recording the measuring result.

NOTE

When the **Autohold** function is activated, the measuring result is automatically displayed permanently as soon as a stable weighing value has been achieved.

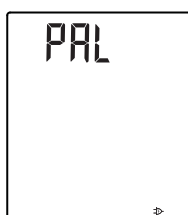
1. Ensure that there is no load on the scale.
2. Ask the patient to step onto the scale.
3. Press the key.
 - ⇒ The display flashes until a stable weight is measured. The measuring result is then displayed permanently.
 - ⇒ The  symbol and the **HOLD** message are displayed.
4. To deactivate the function, press the key.
 - ⇒ The  symbol and the **HOLD** message are no longer displayed.


Entering patient data (input)

You can enter the following patient data directly on the multifunctional display: Age, gender, and physical activity level. The patient data are sent with the measuring results when you send them to a compatible seca product or an EMR system.

1. Press the **input** key.
 - ⇒ The first time this menu is called up after the device starts up, the **PAL** (physical activity level) parameter is displayed.
 - ⇒ If you call this menu up again whilst the device is switched on, the last parameter selected will be displayed.
2. Use the  **send** or  **print** key to select a menu item:

Parameter	Meaning
PAL	Physical activity level
AGE	Age
GEn	Gender

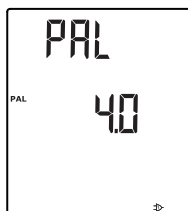3. Confirm your selection.
 - ⇒ The value from the previous measurement is displayed.
4. You have the following options for continuing:
 - Adopt value
 - Change value: Press the  **send** or the  **print** key

Function	Setting
Physical activity level (PAL)	1.0 to 5.0
Age (AGE)	• Up to 3 years in months • Up to 18 years in half-years • From 18 years in years
Gender (GEn)	• Male • Female

5. Press the **input** key.
 - ⇒ The values will be saved.
 - ⇒ You exit the function automatically.
6. If you also want to make settings for **AGE** and **GEn**, repeat the process.

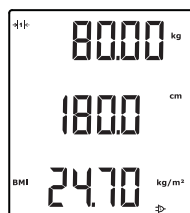
Entering patient's gender

Use the key to enter the patient's gender.

NOTE

This setting overwrites the setting made under \.

Determining Body Mass Index (BMI)

► In order to switch between “male” and “female”, press the key.

The Body Mass Index correlates height and weight.

1. Ensure that the **BMI** calculation method is shown in the multifunctional display
→ [Zwischen BMI und BFR wechseln \(body\)](#), page 76.
2. Start the measurement procedure → [Performing measurement procedure](#), page 70.
⇒ The patient's height, weight and **BMI** are displayed and announced.
3. If weight is to be displayed permanently, press the **hold** key.
⇒ The patient's weight is displayed permanently.

NOTE

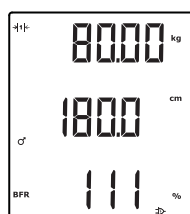
When the **Autohold** function is activated, the measuring result is automatically displayed permanently as soon as a stable weighing value has been achieved.

4. Ask the patient to step off the weighing platform.
5. To prevent out-of-date data leading to faulty results, press the **clear** key.
⇒ Patient data, height, and **BMI** will be deleted.

NOTE

If the **ACLr** function is activated, measured and calculated values will be deleted automatically after 5 minutes.

Determining Body Fat Ratio (BFR)

The Body Fat Ratio correlates height, weight, and gender.

1. Ensure that the **BFR** calculation method is shown in the multifunctional display
→ [Zwischen BMI und BFR wechseln \(body\)](#), page 76.
2. In order to switch between “male” and “female”, press the key.
3. Start the measurement procedure → [Performing measurement procedure](#), page 70.
⇒ The patient's height, weight and **BFR** are displayed.

NOTE

There is no voice output for the **BFR**. Instead, the patient's **BMI** is announced in addition to the height and weight.

4. Press the **hold** key.
⇒ The patient's weight is displayed permanently.

NOTE

When the **Autohold** function is activated, the measuring result is automatically displayed permanently as soon as a stable weighing value has been achieved.

5. Ask the patient to step off the weighing platform.
6. To prevent out-of-date data leading to faulty results, press the **clear** key.
⇒ Patient data, height, and **BFR** will be deleted.

NOTE

If the **ACLr** function is activated, measured and calculated values will be deleted automatically after 5 minutes.

Clearing stored values (clear)

Out-of-date measuring results and patient data lead to faulty calculation of Body Mass Index (**BMI**)/Body Fat Ratio (**BFR**). Use the **clear** key to delete the following measured values and patient data:

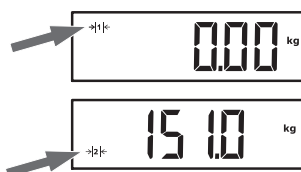
- Height
- **BMI**
- **BFR**
- Gender
- **PAL** (physical activity level)
- Age

NOTE

If the **ACLR** function is activated, measured values and patient data will be deleted automatically after 5 minutes → [Deleting values automatically \(ACLR\)](#), page 76.

- ▶ Press the **clear** key.
 - ⇒ Patient data, height, and Body Mass Index (**BMI**)/Body Fat Ratio (**BFR**) are deleted. ---- is displayed instead.
 - ⇒ The gender symbol goes out.

Switching weighing range automatically

The scale has two weighing ranges. In weighing range 1 (→|1|←), graduations in the weight display are finer. In weighing range 2 (→|2|←), you can use the maximum capacity of the scale.

After the scale is switched on, weighing range 1 is active.

If a particular weight value is exceeded, the scale automatically switches to weighing range 2.

- ▶ To switch back to weighing range 1, completely remove the weight from the scale.
 - ⇒ Weighing range 1 is active again.

Switch off the device

- ▶ Press the  key.
 - ⇒ The multifunctional display goes out.

6.2 Configuring voice output

Selecting language (LAn9)

You can select the language for voice output.

1. Select the item **VOICE** from the menu.
2. Confirm your selection.
3. Select the **LAn9** menu item.
4. Confirm your selection.
 - ⇒ The current setting is displayed.
5. Select the desired setting (in this case: **dE**, German).
6. Confirm your selection.
 - ⇒ The setting is adopted.
 - ⇒ The menu closes.

Setting volume (VOL)

You can adjust the volume of voice output (**0** = off, **100** = max.).

1. Select the item **VOICE** from the menu.
2. Confirm your selection.
3. Select the **VOL** menu item.
4. Confirm your selection.
 - ⇒ The current setting is displayed.
5. Select the desired setting (in this case: Volume **20**).
6. Confirm your selection.
 - ⇒ The setting is adopted.
 - ⇒ The menu closes.

Activating/deactivating announcement of measuring results (reSUL)

You can set the device so that the measuring results (weight, height and **BMI**) are announced after every measurement procedure.

NOTE

If the language setting "English" has been selected, measuring results – in the same way as the display setting – are announced in imperial or in SI units. In all other languages, measuring results are announced in SI units only.

1. Select the item **VOICE** from the menu.
2. Confirm your selection.

3. Select the **reSUL** menu item.
4. Confirm your selection.
⇒ The current setting is displayed.

5. Select the desired setting:
 - **On**
 - **OFF**
6. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Activating/deactivating the announcement "Take your printed ticket" (Print)

If you have connected a seca wireless printer to the device, you can set the device so that following the measurement, there is an announcement telling you to take the results printout.

1. Select the item **VOICE** from the menu.
2. Confirm your selection.

3. Select the **Print** menu item.
4. Confirm your selection.
⇒ The current setting is displayed.

5. Select the desired setting:
 - **On**
 - **OFF**
6. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Activating/deactivating announcement of patient instructions (InStr)

NOTE

Ensure that a language is set that the patient understands → [Selecting language \(LAn9\), page 74](#).

1. Select the item **VOICE** from the menu.
2. Confirm your selection.
3. Select the **InStr** menu item.
4. Confirm your selection.
⇒ The current setting is displayed.

5. Select the desired setting:
 - **On**
 - **OFF**
6. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Configuring beeps

Information on how to configure beeps can be found here: → [Activating/deactivating beeps \(bEEP\), page 78](#).

6.3 Other functions (menu)

Deleting values automatically (ACLR)

Out-of-date measuring results and patient data lead to faulty calculation of **BMI/BFR**. You can set the device so that the measuring results and patient data below are deleted automatically after 5 minutes:

- Gender
- Physical activity level (**PAL**)
- Age
- Height
- **BMI**
- **BFR**

NOTE

If you want to enter **PAL**, age, and gender for the next measurement, the values of the previous measurement will be suggested → [Entering patient data \(input\), page 72](#).

This function is activated at the factory on some models. You can deactivate the function if desired.

A digital display showing the text "ACLR" in a stylized font.

1. From the menu, select the **ACLR** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select the desired setting:
 - **On**
 - **OFF**
4. Confirm your selection.
⇒ The menu closes.
⇒ Patient data, height, and **BMI/BFR** will in each case be deleted 5 minutes after a measurement. ---- is displayed instead.
⇒ The gender symbol goes out.

Deactivating height measurement

You can deactivate height measurement if you just want to use the weighing function.

A digital display showing the text "ULtrA" in a stylized font.

1. From the menu, select the **ULtrA** menu item.
2. Confirm your selection.
3. Select the desired setting:
 - **OFF**
4. Confirm your selection.
⇒ The height measurement is deactivated for the current measurement procedure.
⇒ The menu closes.

NOTE

The height measurement is active again once you switch off the device and then switch it back on again.

Switching between BMI and BFR (body)

You can select whether the measuring station calculates Body Mass Index (**BMI**) or Body Fat Ratio (**BFR**). The current setting is displayed permanently in the multifunctional display.

The calculation is performed automatically as soon as the patient's weight and height have been measured.

A digital display showing the text "body" in a stylized font.

1. From the menu, select the **body** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Press the ▲ **send** or ▼ **print** key to switch between **bMi** and **bFr**.

4. Confirm your selection.
 - ⇒ The setting is adopted.
 - ⇒ The menu closes.
 - ⇒ The setting is displayed in the multifunctional display.

Setting display backlighting (LCd)

You can change the duration and brightness of display backlighting.

1. From the menu, select the **LCd** menu item.
2. Confirm your selection.
3. Select a menu item.
 - Duration (**dUr**)
 - Brightness (**brl**)
4. Confirm your selection.
 - ⇒ The current setting is displayed.
5. Select the desired setting.

Function	Setting
Duration	• SHrt (approx. 15 sec.) • LOnG (approx. 45 sec.) • PErM (permanent)
Brightness	• 50 % • 100 % • OFF

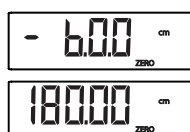
6. Confirm your selection.
 - ⇒ The setting is adopted.
 - ⇒ The menu closes.

Using correction value for height measurement (ZEro)

The **ZEro** function allows you to save a correction value for the height measurement (e.g. a standard height for shoe heels). The device can correct a measuring result with this value, e.g. if a patient is measured fully clothed.

1. From the menu, select the **ZEro** menu item.
 - ⇒ **ZEro** is displayed.
 - ⇒ The last correction value set flashes in the display (with a minus sign in front of it).
2. You have the following options for continuing:
 - Adopt value
 - Change value: Press the **▲ send** or the **▼ print** key
3. Confirm your selection.
 - ⇒ The current correction value is displayed.
4. Ask the patient to step onto the weighing platform.
 - ⇒ The patient's height is displayed.
 - ⇒ The correction value was deducted automatically.
5. To deactivate the function, select the **ZEro** menu item from the menu again.
6. Confirm your selection.
 - ⇒ The correction value is no longer displayed.
 - ⇒ The function is deactivated.
 - ⇒ The menu closes.

Saving additional weight permanently (Pt)

Use the **Pre-Tare** function to save an additional weight permanently and have it deducted from a measuring result automatically. For example, you can save a flat-rate figure for shoes and clothing and then always deduct this from the measuring result if a patient is weighed fully-clothed.

1. From the menu, select the **Pt** menu item.
 - ⇒ The last additional weight set flashes in the display (with a minus sign in front of it).
 - ⇒ The **PT** message is displayed.
2. You have the following options for continuing:
 - Adopt value
 - Change value: Press the ▲ **send** or the ▼ **print** key

NOTE

When you enter the value **0**, the function is switched off. The **PT** message is no longer shown in the display.

3. Confirm your selection.
 - ⇒ The set additional weight is displayed with a minus sign in front of it.
4. Ask the patient to step onto the weighing platform.
 - ⇒ The **PT** message is displayed.
 - ⇒ The patient's weight is displayed.
 - ⇒ The saved additional weight has been deducted automatically.
5. To deactivate the function, select the **Pt** menu item from the menu again.
6. Confirm your selection.
 - ⇒ The set additional weight is no longer displayed.
 - ⇒ The function is deactivated.
 - ⇒ The menu closes.

Activating Autohold (AHOLD) function

When you activate the **Autohold** function, the measuring result for every weighing operation is displayed for every weighing procedure after the weight has been removed from the scale. It is then no longer necessary to activate the **Hold** function manually.

NOTE

Regardless of the setting selected here, the weight of the child is always determined by **Autohold** in the **2 in 1** function.

1. From the menu, select the **AHOLD** menu item.
2. Confirm your selection.
 - ⇒ The current setting is displayed.
3. Select the desired setting:
 - **On**
 - **OFF**
4. Confirm your selection.
 - ⇒ The setting is adopted.
 - ⇒ The menu closes.

Activating/deactivating beeps (bEEP)

You can activate beeps for a number of device functions:

1. From the menu, select the **bEEP** menu item.
2. Confirm your selection.
3. Select a menu item:
 - **PrESS**: beep when key is pressed
 - **HOLD**: beep when weighing value is stable
 - **MEASr**: Beeps during the height measurement
4. Confirm your selection.
 - ⇒ The current setting is displayed.

5. Select the desired setting:
 - **On**
 - **OFF**
6. Confirm your selection.
 - ⇒ The setting is adopted.
 - ⇒ The menu closes.

Setting filtering (FIL)

Setting the filter (**FIL** = filter) allows you to reduce interference when determining weight. The selected setting affects the sensitivity with which the weight display reacts to patient movements and the time until the **Hold** function displays a weight value permanently (**Hold** value).

Setting (FIL)	Sensitivity	Determination of Hold value
0	Sensitive	Slow
1	Medium	Medium
2	Insensitive	Fast

NOTE

With the **0** setting and patients who are not very steady on their feet, it is possible that no weight value is displayed permanently despite the **Hold** function being activated.

The greatest deviation between weight value displayed and actual value occurs at setting **2**.

FIL

1. From the menu, select the **FIL** menu item.
2. Confirm your selection.
 - ⇒ The current setting is displayed.
3. Select a filtering level.
4. Confirm your selection.
 - ⇒ The setting is adopted.
 - ⇒ The menu closes.

Non-verified models: Switching weight unit (Unit)

You can select the unit in which weight is to be displayed.

WARNING! **Patient hazard**

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (weight: kilograms/grams, height: meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- Only use measuring results in SI units.
- The user takes sole responsibility for the use of measuring results in non-SI units.

Unit

1. From the menu, select the **Unit** menu item.
2. Confirm your selection.
 - ⇒ The current setting is displayed.
3. Select the unit in which weight is to be displayed:
 - Kilograms/grams (**gr**)
 - Pounds (**lbs**)
4. Confirm your selection.
 - ⇒ The setting is adopted.
 - ⇒ The menu closes.

Switching unit of height (HUnit)

You can select the unit in which height is to be displayed.

WARNING! **Patient hazard**

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (weight: kilograms/grams, height: meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- Only use measuring results in SI units.
- The user takes sole responsibility for the use of measuring results in non-SI units.


HUnit

1. From the menu, select the **HUnit** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select the unit in which height is to be displayed:
 - **FEET**: Feet, inches
 - **Mtr**: Centimeters
4. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Restoring factory settings (rESet)

You can restore the factory settings for the functions below:

Function	Factory setting
Autohold (AHOLd)	On
Beep (PrESS)	On
Beep (HOLd)	On
Filtering (FIL)	0
Autoclear (ACLR)	On
Pre-tare (Pt)	0 kg
Unit, weight (Unit)	kg (gr)
Display backlighting (brightness brl)	50 %
Display backlighting (duration dUr)	Permanent (PErM)
Body Mass Index (BMI)/Body Fat Ratio (BFR)	Body Mass Index (bMi)
Physical activity level (PAL)	1.0
Age in years	18
Age in months	0
Height measurement (ULtrA)	On
Beep (MEASr)	On
Language, voice output (LAn9)	Variant-depen- dent
Volume (VOL)	50 %
Announcement of patient instructions (InStr)	On
Announcement of measuring results (MEASr)	Variant-depen- dent
Announcement "Take your printed ticket" (Print)	OFF

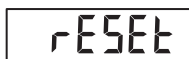1. From the menu, select the **rESEt** menu item.
2. Confirm your selection.
⇒ The menu closes.
3. Switch off the scale.
⇒ Factory settings are restored and are available when the scale is switched on.

7 HYGIENE TREATMENT

WARNING!

Electric shock

The device is not de-energized when the on/off button is pressed and the display goes out. Use of fluids on the device may cause an electric shock.

- ▶ Before each hygiene treatment, ensure that the device is switched off.
- ▶ Disconnect the power supply connector before each hygiene treatment.
- ▶ Ensure that no fluids penetrate the device.

WARNING!

Risk of infection

- ▶ Subject the device to a hygiene treatment at regular intervals as described in this section.

NOTICE!

Damage to device

Unsuitable cleaning agents and disinfectants may damage the sensitive surfaces of the device and lead to transparent components becoming clouded.

- ▶ Do not use aggressive or abrasive cleaning agents.
- ▶ Do not use organic solvents (e.g. white spirit or petroleum spirit).
- ▶ Ensure that no moisture or dust get into the sensors during cleaning.

7.1 Cleaning

- ▶ If required, moisten a soft cloth with a mild soap solution and wipe the device over with it.

7.2 Disinfecting

1. Disinfect the device at regular intervals with a disinfectant suitable for sensitive surfaces and acrylic glass (e.g. 70 % ethanol).
2. Follow the instructions for use for the disinfectant.
3. Disinfect the device:
 - ▶ Moisten a soft cloth with disinfectant and wipe down the device with it.
 - ▶ Comply with the intervals, see table:

Interval	Component
Before each measurement	Weighing platform
After each measurement	Weighing platform
As required	• Multifunctional display • Column • Ultrasound head

7.3 Sterilizing

The device must not be sterilized.

8 FUNCTION CHECK

- Perform a function check before each use.

A complete function check includes:

- A visual inspection for mechanical damage
- A test of the alignment of the device
- A visual and function check of the display elements
- A function check of all the controls shown in the section entitled “Overview”
- A function check of the optional accessories

If you find faults or deviations during the function check, first try to remedy the fault with the aid of the “Troubleshooting” section in this document.

CAUTION!

Personal injury

If you find faults or deviations during the function check which you are unable to remedy with the aid of the “Troubleshooting” section in this document, you must not use the device.

- Have the device repaired by seca Service or an authorized service partner.
- Follow the section entitled “Servicing” in this document.

9 TROUBLESHOOTING

NOTICE!

Malfunction

If the device is equipped with the optional **seca 452** external interface module and is connected to the **seca connect 103** software, error messages and error symbols may occur which are not described here.

- Observe the **seca 103/452** system instructions for use.

If faults occur when operating the device, first attempt to remedy them yourself using the following table. If the fault persists, contact seca Service.

Fault	Cause	Remedy
After calibrating the height measurement, the FAIL message appears.	Calibration of the height measurement has failed.	Repeat calibration (→ Calibrating height measurement, page 69): • Using supplied calibration rod. • Place the calibration rod in the middle of the foot pictogram of the weighing platform. • Ensure that there are no objects or people in the immediate vicinity of the device (→ Setting up device, page 66).
No weight is displayed even though there is a load on the scale.	The device has no power supply.	• Ensure that the power supply unit is connected. • Switch on device.

Fault	Cause	Remedy
Status LED does not come on.	Connector cable in the ultrasound measuring head has not been plugged in properly.	Ensure that the connector cable has been plugged in properly.
	Ultrasound measurement is deactivated.	Switch off the device, then switch it back on again.
	Status LED is faulty.	Inform seca Service.
Patient instructions are not announced.	Function is deactivated.	Activate function (→ Activating/deactivating announcement of patient instructions (Instr) , page 75).
	Ultrasound measurement is deactivated.	Switch off the device, then switch it back on again.
	Volume set to zero.	Increase volume (→ Setting volume (VOL) , page 74).
	Loudspeaker faulty.	Inform seca Service.
No beeps audible.	Function is deactivated.	Activate function (→ Activating/deactivating beeps (bEEP) , page 78).
	Ultrasound measurement is deactivated.	Switch off the device, then switch it back on again.
	Volume set to zero.	Increase volume (→ Setting volume (VOL) , page 74).
	Loudspeaker faulty.	Inform seca Service.
Measuring results are not announced.	Function is deactivated.	Activate function (→ Activating/deactivating measuring results (reSUL) , page 74).
	Ultrasound measurement is deactivated.	Switch off the device, then switch it back on again.
	Volume set to zero.	Increase volume (→ Setting volume (VOL) , page 74).
	Loudspeaker faulty.	Inform seca Service.
Request to take the results printout is not announced.	Function is deactivated.	Activate function (→ Activating/deactivating the announcement "Take the results printout" (Print) , page 75).
	Ultrasound measurement is deactivated.	Switch off the device, then switch it back on again.
	Volume set to zero.	Increase volume (→ Setting volume (VOL) , page 74).
	Loudspeaker faulty.	Inform seca Service.
0.00 is not displayed before weighing takes place.	A load was already on the device before it was switched on.	- Remove the load from the device. - Switch off the device, then switch it back on again.
----- is displayed before weighing takes place.	A load was already on the device before it was switched on.	- Remove the load from the device. - Switch off the device, then switch it back on again.
The weight display flashes.	Consistent weight value not yet detected.	- One of the following functions is active: Hold, Autohold, Tare. Wait until the device has detected a consistent weight value. The display stops flashing. - Take the load off the device, wait until 0.00 is displayed and repeat the measurement procedure.

Fault	Cause	Remedy
Implausible measuring results.	• Patient moved too much during the measurement. • Patient did not stand upright during the measurement.	• Take the load off the device, wait until 0.00 is displayed and repeat the measurement procedure. • With activated Autohold function: Press the hold key which restarts the measurement procedure.
One display segment is displayed continuously or not at all.	The display is defective.	Inform seca Service.
StOP is displayed.	Maximum capacity has been exceeded.	Remove the load from the device.
tEMP is displayed.	The ambient temperature is too high or too low.	• Operate the device only in the intended ambient conditions (→ General technical data, page 86). • After transporting or storing the device, wait about 15 minutes before switching it on.
Multifunctional display no longer reacts when keys are pressed.	Key lock is active.	Switch off key lock (→ Activating/deactivating key lock, page 71).
	Device is in an undefined state following implausible input.	• Take power supply unit out of the power supply socket. • Wait approx. 1 minute. • Plug the power supply unit into the power supply socket; the scale and the multifunctional display will switch on automatically.
	The keypad is defective.	Inform seca Service.
Er:6:80 is displayed.	Voice output memory cannot be read.	Inform seca Service.
Er:6:81 is displayed.	Voice file not found.	Temporarily deactivate beeps and voice output to suppress error messages: • → Activating/deactivating announcement of measuring results (reSUL), page 74. • → Activating/deactivating announcement of patient instructions (Instr), page 75. • → Activating/deactivating beeps (bEEP), page 78.
Er:6:82 is displayed.	An error occurred during the measurement procedure.	Ask the patient to keep still and repeat the measurement procedure.
		If the error message recurs: Inform seca Service.
Er:6:83 is displayed.	Interference caused by reflections	• Remove objects from the immediate vicinity of the device. • Ask people to stay farther away from the device. • → Setting up device, page 66
	Interference caused by other ultrasonic emitters	Increase the distance from other ultrasonic emitters.
Er:6:84 is displayed.	Ambient temperature is too high or too low.	• Operate the device only in the intended ambient conditions (→ General technical data, page 86). • After transporting or storing the device, wait about 15 minutes before switching it on.
	Temperature sensor is defective.	Inform seca Service.

10 SERVICING

10.1 Weight measurement, verified models

seca recommends having your device serviced prior to verification.

CAUTION!

Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find a service partner in your vicinity at www.seca.com or by emailing service@seca.com.

CAUTION!

Faulty measurements as a result of verification being performed poorly or not at all

- ▶ Have verification performed only by authorized persons.
- ▶ Always have verification performed if one or more quality seals are damaged.

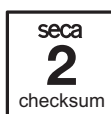
Have authorized persons perform verification in line with national legal regulations.

Verification is necessary whenever one or more quality seals are damaged or the contents of the verification counter no longer match the number on the applicable verification sticker. If quality seals are damaged, contact seca Service directly.

Verifications may only be performed by authorized agencies. To guarantee this, the scale is equipped with a verification counter which records each change in verification-related data.

If you want to check whether the scale has been properly verified, proceed as follows:

1. Ensure that the device is switched off.
2. Press the  key of the multifunctional display.
⇒ All the elements of the display are shown briefly.
3. Press any other key on the multifunctional display (not the  key again) whilst all the elements of the display are shown briefly.
⇒ The  symbol is displayed.
⇒ The current content of the verification counter flashes in the display for a few seconds.
4. Compare the content of the verification counter displayed with the number on the verification counter sticker.

Both numbers have to match for the verification to be valid. If the verification sticker and the verification counter do not match, the scale must be verified. Please contact your service partner or seca Service. Once the scale has been verified, a new, updated verification counter sticker is used to identify the verification counter reading. The person authorized to perform verification secures this verification with an additional seal. The verification counter sticker can be ordered from seca Service.

10.2 Weight measurement, non-verified models

The product must be set up carefully and serviced regularly. Depending on how frequently the scale is used, seca recommends servicing at intervals of 3 to 5 years.

CAUTION!

Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find a service partner in your vicinity at www.seca.com or by emailing service@seca.com.

10.3 Height measuring devices

The product needs to be set up carefully and serviced regularly. Depending on how frequently the product is used, seca recommends servicing at intervals of 3 to 5 years.

CAUTION!

Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find a service partner in your vicinity at www.seca.com or by emailing service@seca.com.

11 TECHNICAL DATA

11.1 General technical data

seca 287/seca 286	
Dimensions • Depth • Width • Height (standard model)	466 mm 435 mm 2251 mm
Weight of measuring station	Approx. 16.5 kg
Ambient conditions, operation • Temperature • Air pressure • Humidity	+10 °C to +40 °C / +50 °F to 104 °F 700 – 1060 hPa 30 % – 80 %, no condensation
Ambient conditions, storage • Temperature • Air pressure • Humidity	-10 °C to +65 °C / +14 °F to 149 °F 700 – 1060 hPa 0 % – 95 %, no condensation
Ambient conditions, transport • Temperature • Air pressure • Humidity	-10 °C to +65 °C / +14 °F to 149 °F 700 – 1060 hPa 0 % – 95 %, no condensation
Power supply Power supply unit • Supply voltage • Maximum current consumption • Insulated device, protection class II (EN 60601-1)	12 V Typically 500 mA 

17-10-07-671-100b_2024-11S

seca 287/seca 286	
Power supply voltage	100 V – 240 V
Power supply frequency	50 Hz – 60 Hz
Current consumption (display backlighting off, volume 50 %)	Approx. 100 mA
Medical device in line with directive (EU) 2017/745	Class I with measuring function
EN 60601-1: Medical electrical equipment, type B	
Type of protection	IP20
Duty cycle	Continuous duty
Application parts in line with IEC 60601-1	Multifunctional display, glass plate
Minimum weight (to trigger measurement procedure)	0.5 kg

11.2 Weight measurement

Non-verified model	
Maximum capacity	300 kg
Minimum capacity	1 kg
Graduation	50 g
Tare range	300 kg (subtractive)
Accuracy • 0 kg to 33 kg • 33 kg to 300 kg at 20 °C ambient temperature and 45 % humidity	± 50 g ± 0,15 %

Verified model	
Verification in line with Directive 2014/31/EU	Class III
Maximum capacity • Weighing range 1 • Weighing range 2	150 kg 300 kg
Minimum capacity • Weighing range 1 • Weighing range 2	1 kg 2 kg
Graduation • Weighing range 1 • Weighing range 2	50 g 100 g
Tare range	300 kg (subtractive)
Accuracy on initial verification • Weighing range 1: 0 to 25 kg • Weighing range 1: 25 to 100 kg • Weighing range 1: 100 to 150 kg • Weighing range 2: 0 to 50 kg • Weighing range 2: 50 to 200 kg • Weighing range 2: 200 to 300 kg	± 25 g ± 50 g ± 75 g ± 50 g ± 100 g ± 150 g

11.3 Height measurement

Measuring range, graduation, accuracy	
Measuring range	60 – 210 cm
Graduation	1 mm
Accuracy (measuring range 100 to 200 cm) at 20 °C ambient temperature, no air movement, no interfering objects in the vicinity of the measuring range	± 5 mm

Signals and voice output	
Power LED on ultrasound head lights up continuously.	The device is ready to measure.
"Please stand upright and look straight ahead."	Instruction to the patient.
Power LED on ultrasound head goes off.	The measurement is in progress.
"Do not move. The measurement starts now."	Instruction to the patient.
Short beeps.	The measurement is in progress.
Long beep.	The measurement is complete.
"Your weight is (...) kilograms. Your height is (...) centimeters. Your BMI is (...)."	Announcement of the measuring results.
"The measurement is complete. Please leave the platform."	Instruction to the patient.

12 OPTIONAL ACCESSORIES AND SPARE PARTS

Accessory/spare part	Article number
Switch-mode power supply: 100 V – 240 V~ / 50 Hz – 60 Hz, 12 V= / 0.5 A	68 32 10 270
Accessories for integration via WiFi/LAN • seca 452 external interface module • seca connect 103 software • Barcode scanner (medical device)	452 0000 009 Will be provided in the course of integration projects Cannot be ordered from seca; for recommendations, see system instructions for use for seca 103/452
Privacy cover for multifunctional display	01 13 05 503

13 DISPOSING OF THE DEVICE

Do not dispose of the device in your household waste. The device must be properly disposed of as electronic scrap. Follow your respective national regulations. For more information, please contact seca Service or an authorized service partner.

14 WARRANTY

There is a two-year warranty period from delivery for defects attributable to poor materials or workmanship. All movable parts, e.g. batteries, cables, power supply units, rechargeable batteries etc. are exempt. Defects which come under the warranty will be repaired for the customer free of charge against proof of purchase. Additional claims cannot be considered. Costs of transport to and from seca are the responsibility of the customer if the device is located somewhere other than the customer's headquarters. In the event of transport damage, claims under warranty can only be made if the complete original packaging was used for transport and the device was secured and fastened in it according to its originally packaged condition. You should therefore keep all packaging parts.

The warranty will be voided if the device is opened by persons not expressly authorized by seca to do so.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

15 DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product complies with the terms of the applicable European directives and regulations. The unabridged declaration of conformity can be found at www.seca.com.

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